



STERILISING SERVICE



www.podmedsupplies.com.au

Ph: 1300 358 064

HOW YOU RECEIVE THE ORDER

- » You will receive the sterile instruments inside the 'Clean Sterile Instrument Container'
- » A 'Used Instrument Container' is packed as well- this has been thoroughly cleaned. This provides extra protection during shipping
- » Keep the shipping box so it can be sent back once instruments are used





HOW YOU RECEIVE THE ORDER

- » There are two boxes with two lids.
- » Clean instrument light coloured box- with compliance and instrument details provided.
- » Used instrument darker coloured box with BIOHAZARD label on front.

CLEAN INSTRUMENT BOX

Instrument packs contains the following information on for your records.

- Sterilisation date

- QR code for compliance information


- Expiry date

- Printex indicator label- further proof of sterilisation

Keep instruments inside this box till they are ready to be used. If transporting instruments, ensure the lid is firmly attached.



PACKS OF INSTRUMENTS

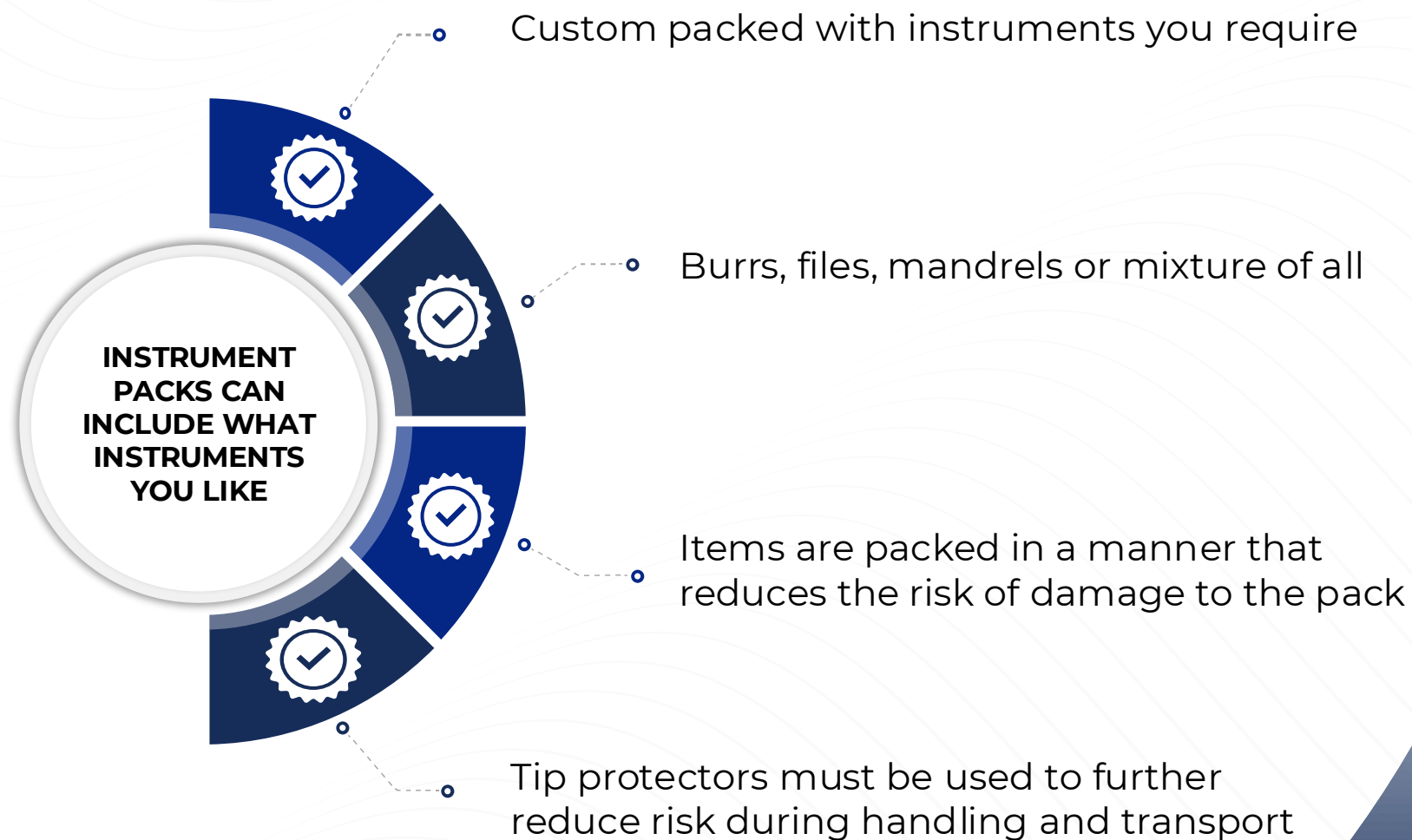



USED INSTRUMENT BOX

- ▶ After use, items must be cleared of any visible debris and bodily fluids by wiping them clean or rinsing thoroughly under running water.
- ▶ If you are transporting this box, ensure the lid is firmly attached.



CAUTION!
USED MEDICAL INSTRUMENTS
 Please pack in accordance to instructions provided

Date of Document:

Clinic Name:

Clinic Address:

Clinic Phone: Clinic Contact:

Returned Items:

Item	Count	Total

Total items:

Clean boxes returned:

Dirty boxes returned:

Name: Signed:

Notes

Processed by PodMed Supplies P.O Box 260 Daylesford Victoria 3460
 ☎ 1300 358 064 ✉ hello@podmedsupplies.com.au

USED INSTRUMENT BOX

01

Mark the return sheet each time you add used instruments to this container to keep an accurate count of items used and to be returned

02

Ensure this page is completed accurately and return it with the used instruments by placing the return sheet on top of the used instrument box for shipping

CAUTION



BIOHAZARD

**Used Medical Instruments
Handle With Care**

Scan QR code to order more sterile instruments. Ensure pick up sheet is filled out 

PodMed Supplies
P.O Box 260
Daylesford Victoria
Ph: 0421 449 368

 Scan QR code to arrange collection. Ensure pick up sheet is filled out

USED INSTRUMENT BOX

Once the required number of sets have been used and placed in the used instrument box, please scan the QR code to notify us that the instruments are ready for collection

Packing Shipping Box

- » The lid of the 'clean instrument box is to be placed outer side facing down first- under the used instrument box
- » Place the used instrument box inside the empty clean container.
- » Attach the BIOHAZARD lid on then tape down.
- » Place both inside the shipping box. Place the returned item sheet on top of the BIOHAZARD label.



RETURN OF INSTRUMENTS

- » Close the box and tape to secure.
- » Fix supplied return overtop of the label used to ship to you (if using AusPost) otherwise a label will be emailed to you.
- » If courier pick up is available let us know and we can arrange collection.
- » If courier pick up is not available please lodge at any AusPost outlet- No payment required.





THANK YOU